

Lumify iOS | 5.0 Patient Data Transfer

Global Education Services Ultrasound Services and Solutions Deliverability 2023

Before we get started

Connectivity Profiles

Non-default settings and customization selections do not transfer between Lumify 2.0 and 5.0 applications. All customized settings must be selected again on the new 5.0 application.

DICOM Services

Lumify 2.0 and Lumify 5.0 use the same port for DICOM services. If both applications are running concurrently during set-up a "Failed to start DICOM Server" message will appear. Select "Continue" to ignore this message.



Lumify Power Modules

Do not plug the Lumify Power Module into the device during the data back up or transfer process. Doing so may automatically launch the Lumify 2.0 application.





Lumify 2.0

SSD Education Services

 \rightarrow

1. Open the application on the device by selecting the green Lumify 2.0 icon.



2. The Lumify application will launch and open the home screen.



3. Open the menu by touching the hamburger menu (three horizontal lines) in the upper left corner of the screen.



4. Select Settings.



5. Select Patient Database.



6. Select Export Database.

This menu is where patient database file names are entered, and their folder destination is selected.



7. Follow the password requirements. Enter and confirm a password to encrypt the database. This action is required to move forward.



8. Review the safety prompt and press Okay.



9. The export will begin, and a pop-up will automatically close upon completion. The data transfer time will vary depending on the size and number of exams in the database.

19:39 😚		all 🗢 1 17
	Export	Export
L	umify Patient Data	
1. Please entr database exp 2. Enter a Pa in order to er exnort 3. Export 1. Constant 2. 2. 3. 3. 5. 5. 5. 1. 1. 1. 1. 1. 1. 1. 1	er a Filename for th sort ssword and Confirr rcrypt the patient of Progress valt	ne patient n Password, Jatabase alient the panent the panent the said include

10. When the export is complete the Export menu will be visible. Select the export destination.



Note: Patient data can be sent to the new Lumify 5.0 application by saving the data directly in the files on the device or it can be transferred to another device.

SSD Education Services

11. To save the data to the same device, select **Save to Files**.



12. A default destination folder will appear. Select the default destination.



Note: Users can also create a new destination or an existing destination folder.

13. To verify the export saved to the correct destination go to the home screen and select the **Files** app.



14. If the Patient Data files are not located in the Files app, browse the device and select **On My iPhone**.

14:3 ◄ Search	81	.ul 🗢 🗷
		\odot
Bro	owse	
Q Se	earch	
Loc	ations	~
	On My iPhone	>
	iCloud Drive	>
	OneDrive Personal	>
団	Recently Deleted	>
Fav	ourites	~
J	Downloads	>
Тад	js	~
•	Green	>
•	Orange	>
•	Yellow	>
	Gray	>
Ref	cent Shared	Browse

SSD Education Services

15. Select the green Lumify 2.0 folder. All exams that have been successfully imported will be visible.





1. Select the blue Lumify 5.0 folder.



2. The application will launch. Review the data migration alert and press **Close** to proceed.



3. Open the menu by touching the hamburger menu (three horizontal lines) in the upper left corner of the screen.



4. Select Settings.



SSD Education Services

5. Select Patient Database.

23:40	LTE 78 9
< Settings	
Imaging	>
Display	>
Patient Database	>
Logs	>
Customer Information	>
Device	>
Registered Transducers	>
Lumify Power Module	>

6. Select Import Database.



SSD Education Services

7. In the Import Database menu, click **Select a file**.



8. Most devices will launch the **Recent** directory by default. Within the **Recent** directory the data files will be visible.

Select the file then select **Open**.

14:51		.ıl 🗢 69		14:53		내 수 69
	Recent	Cancel			Recent	BBOpen
Q Search				Q Search		
Lumify Patienta.bak 14:01 22 KB		_	1 →	Lumify Patienta.bak 14:01 22 KB		
Recent	Shared	Browse		Select All		Unselect All

SSD Education Services

9. If data is not visible in the **Recent** directoy, select the **Browse** icon on the bottom right of the screen.

14:51		.ıl 🗟 🚳
	Recent	
Q Search		
	=0	-
Recent	Shared	Browse
		-

10. Select On My iPhone/iPad

14:3 ◄ Search	31	.ıl 🗢 🔃
Bro	owse	
Q Se	earch	
Loc	cations	~
	On My iPhone	>
	iCloud Drive	>
	OneDrive Personal	>
Ŵ	Recently Deleted	>
Fav	vourites	~
€	Downloads	>
Тас	js	~
•	Green	>
٠	Orange	>
•	Yellow	>
	Gray	>
Re	cent Shared	Browse

11. The export destination folder will be visible. In this example, the Lumify 2.0 Folder is the specified destination folder. Select the folder.



12. The patient data file will be visible. Select the file then select **Open**.



13. The data file selected will be visible in the **Database File** field.



14. Enter the password chosen during the export process in the **Password** field. Select **Import** on the top right of the screen.



15. Select **Confirm**. The import process will begin.

Note: The file size will impact the duration of the import process.



16. Upon completion of the import, an **Import Summary** prompt will appear.

Select Okay.



17. The application will redirect to the Patient Database menu.

To return to the previous **Settings** menu, select the arrow on the top left of the screen.

Patient Database Repair epair Database Repair tempts to repair potential conjuption that has occurred in chatabase. Imaging eset Database Reset eletes all patient data and resets the patient database. Display wordt Database Export repatient database to a single file for archive or import. Customer Information Device Device
Repair Database Repair ttempts to repair potential corruption that has occurred in re database. Imaging Reset Database Reset teletes all patient data and resets the patient database. Patient Database Export Database Export ave patient, database to a single file for archive or import. Device
Attempts to repair potential conjuption that has occurred in the database Display Reset Database Reset Deletes all patient data and resets the patient database. Logs Export Database Export Save patient database to a single file for archive or import Device
Reset Database Reset Deletes all patient data and resets the patient database. Logs Export Database Export Save patient database to a single file for archive or import. Device
Deletes all patient data and resets the patient database. Export Database Export Save patient database to a single file for archive or import Device
Export Database Export Save patient database to a single file for archive or import Device
Save patient database to a single file for archive or import Device
on another device.
Import Database Import Registered Transducers
Import a patient database file.

18. Select the arrow on the top left to return to the home screen.



19. To verify the upload was successful, select the hamburger menu on the top left of the screen.



20. Select Saved Exams.



21. All exams that have been successfully imported will be visible.



22. Remove the Lumify 2.0 application from the device.

Note: The steps to remove applications varies based on the version of iOS running on the device. Refer to the <u>Apple Support iPhone User Guide</u> to access detailed, version-specific instructions.

©2023 Koninklijke Philips N.V. All rights are reserved.



www.philips.com.